

LUBRICATING AND PROTECTIVE SOLUTION

HOW SUPPLIED

Spray bottle containing 20 ml of solution

COMPOSITION

• Hydroxygenated triglycerides composed of vegetable essential fatty acids ... including linolenic acid, oleic acid, stearic acid.	92%
• Excipients: fish essential fatty acids	5 %
• Fragrance	3 %

PROPERTIES

ULCILENE® restores the hydro-lipid protective film of the skin, limits insensible perspiration, facilitates micro-perfusion of the underlying skin and promotes epidermal cell turnover. Light, gentle massage with ULCILENE® is painless. ULCILENE® contains tocopherols and phytosterols (vitamins A, E, etc.).

INDICATION

ULCILENE® is indicated for the treatment of redness at pressure points and the prevention of pressure ulcers on high-risk zones.

INSTRUCTIONS

- 2 to 3 applications per day, by light, gentle massage.
 - Apply the product lightly with the fingertips for one minute.
- Repeat the operation on each zone to be treated as often as necessary (an average of 2 to 3 times a day).

PRECAUTIONS FOR USE

The safety and acceptability of this product have been demonstrated. In the case of allergic reactions, stop using the treatment and seek medical attention.

Oxygen therapy: it is not recommended to use liquid oxygen in the presence of fats.

Does not contain a driving gas.
Keep out of the reach of children.



at temperatures
greater 315°C.

Store protected from light and heat.

Date of last revision : June 2015

FOR EXTERNAL USE

Bottle **20ml e**

Medical device complying with the requirements of Directive 93/42/EEC.

Distributor in Lebanon

C.T.R Museum district – Badaro – P.O. Box 11-0525 – BEIRUT - LEBANON

Name and information of the manufacturer :



Laboratoires Carliène
78360 Montesson - France



Medical Device in conformity with the requirements of the Directive 93/42/CEE & 2007/47/CE



Consult the information leaflet



Batch n°



Expiration date